

Manual Connect Offroad
Udinummer 870261760Coo97899975321YL



Features Connect Offroad

- The Connect Offroad is the ideal solution for the wheelchair when you go off the road and in bad weather conditions in winter (snow).
- Very easy to connect and disconnect by means of an easy and reliable connection.
- For folding wheelchair: adapter always removable.
- The Connect Offroad is made of high-quality stainless steel. Weight: ± 5 kg.
- Warranty: parts 1 year / frame 5 years.
- Delivery time: 5 working days after receipt of the wheelchair. installation directly by appointment is also possible.



ROAM Special Cycles B.V.
Buitenkamp 20
5321KB Hedel
Tel.nr. 073-5997042
E-mail: info@roam.nl
Internet: www.roam.nl

1 January 2023



Introduction

Roam is a manufacturer of unique and special "bicycles" whereby independency of users is priority number one.

The Roam Connect Offroad is available in both junior as well as a senior version and can easily be connected to any wheelchair.

Roam was founded in 1987 by Theo de Rouw, who started doing business - on his own initiative - in the evenings. His goal was: independency of disabled people as much as possible.

As a result of this objective the business grew into a full-time job in less than 3 years. With the same objective in mind, the company was converted into a V.O.F. in 1997 and wife Johanna de Rouw became one of the partners. Later on Johan de Rouw joined the company.

Over the years until now independency of disabled people has still remained the most important keyword for Roam.

The management director is Johan de Rouw and the company has 12 staff members. The company is NEN-EN-ISO 9001:2015 and NEN-EN-ISO 13485 certified and works according to the National Care Standards. These certificates ensure that Roam delivers quality - from development and design to delivery and aftercare- .

Roam is a custom friendly company. Her product range is subject to continuous improvement in relation to regularly compliance and user's wishes and needs.

In 2003, Roam was proclaimed: the most creative entrepreneur of the year by the Chamber of Commerce (the certificate was awarded by the Chamber of Commerce). The Roam approach now enjoys international prestige.



Welcome to the Roam-team

Thank you for purchasing our product, the Connect Offroad.

With great care, the Roam team has made your Connect Offroad roadworthy. The user manual has already been received and describes how the Connect Offroad hand bike should be handled. Please note that the complete manual should be read. Roam is not responsible for incorrect use and ignoring the commandments / warnings. In case of incorrect or improper use of the hand bike or manual Roam cannot be held responsible. In other words, as a user you must follow the commandment described in the Manual.

The adventure of freedom and a step towards independence can begin with the Connect Offroad and that means: first getting acquainted with this beautiful product; follow the user manual. Take your time to experience!

The Connect Offroad is made to serve you as well as possible and we hope we enjoy your bike. Do not hesitate to make your findings known to your dealer or the manufacturer.

We wish you a lot of fun with the Roam Connect Offroad.

ROAM Special Cycles B.V.
Buitenkamp 20
5321KB Hedel
Tel.nr. 073-5997042
E-mail: info@roam.nl
Internet: www.roam.nl



Index

Introduction 2

Welcome to the Roam Team 3

Table of Contents 4

Inspection and Delivery 5

Commandments and warnings 6/7/8

**Coupling and uncoupling of the
Connect off Road 8**

Maintenance and your professional dealer 9

Maintenance 9

Tools intended for dealer/Roam Specialist 9

Checklist 10

Tightening moments 10

Warranty 11

EU Declaration of Conformity 12



Delivery check

Check that the delivery of the Connect Offroad matches the packing slip. If transport damage is detected upon delivery, please inform your dealer. Roam trusts that the Connect Offroad meets your expectations and wishes you a lot of success and above all a lot of cycling pleasure.

Important

Upon delivery of your Connect Offroad all parts described on the attached CD are tailor made to your wishes. Certain parts need to be readjusted after some time. The best thing to do is making an appointment with your dealer.



Commandments and Warnings Always observe the commandments and warnings below!

General

When purchasing the Connect Offroad, the dealer must provide the correct information to ROAM Special Cycles BV about the use and application.

Roam cannot be held liable for defects, damage, unsafe actions or situations, accidents or incidents due to incorrect use, incorrect information or incorrect maintenance or repair.

The user of the Connect Offroad must be in good mental and physical health and not be restricted in a sitting position.

The user's hearing and vision must be suitable for the use of the Connect Offroad.

Changes

No changes to the Connect Offroad No changes may be made to the Connect Offroad. Roam is not responsible for damage caused to the Connect Offroad by making changes. Roam cannot be held liable for incorrect use or incorrect information for the application of the Connect Offroad and the resulting damage, unsafe situations / actions, accidents or incidents. The speed of our products may not be increased or changed in any way. When changing our product, the manufacturer is in no way responsible or liable.

User

1. User should read and understand the manual carefully.
2. User is responsible and liable for proper use, as stated in this manual.
3. It is forbidden to use the Connect Offroad under the influence of alcohol /drugs or mind-altering medications.
4. If the user is limited to such an extent that he/she has vision/hearing problems and/or cannot read and understand the manual, parents/guardians/supervisors are responsible for the application of the manual and the use of the Connect Offroad by the end user.
5. Before you start using the Offroad, always check whether the adapter under the wheelchair is properly secured. In the case of detachable adapters, the star buttons or quick-release fasteners must be checked to see if they are properly secured.
6. The Connect Offroad is personal Care when wheeling that you are well equipped (mobile phone).
7. Provide good further equipment such as: repair kit tires, small first aid kit, water bottle with drinking water, some ransom and the identity card.
8. On longer journeys, protect yourself from bad weather conditions in winter or excessive heat in the summer period.
9. In case of fog, bad weather or dark conditions (evening) we advise you to wear an illuminated vest, with signaling.
10. Before going off-road, a helmet must be worn.

Regulations

The regulations for pedestrians apply to the use of the Connect Offroad

Use of the Connect Offroad

1. Right-angled and sharp turns should be taken at an appropriate walking speed. Adapt to the circumstances. All this to prevent the risk of tipping over.
2. For maintenance, settings, adjustments, replacement of parts and / or repairs, reference is made to the dealer / Roam specialist.
3. The Roam specialist as well as the dealer are obliged to follow the manual and to apply the recommended tools.
4. On roads where extra attention is required such as roads with obstacles, along waterways, equipped with elevations, railway crossings, tram tracks or pedestrian crossing and or driving along busy roads etc, should be towed at walking speed.
5. Note that with prolonged hooping, your hands can become warm and blisters or burns can occur. Note that when descending a slope, the speed increases.

Childseat in combination with the Offroad

1. If you use the Connect Offroad in combination with a child seat, you must also put the small wheels on the parking brake when clicking on the child seat in question.
2. This also applies to disconnection.
3. Remember to attach the baby / child with the seat belt.
4. Before you put the baby/child in the child seat, the child seat must first be clicked on the Connect Offroad.
5. Before the child seat is disconnected, the baby / child has first be removed from the seat.
6. Furthermore, we take you to the manual / application guide of the child seats and the instruction videos on our website www.:Roam.nl



Connecting and disconnecting the Connect Offroad

Connect

Connecting the Connect Offroad under your wheelchair is very easy. First of all, you put the wheelchair on the brakes; then close the lever of the connector on the Connect Offroad (down) and now slide the Connect off Road with the bearing holder as far as possible into the adapter of your wheelchair. Now you need to open the lever of the connector on the Connect Offroad (up) so that the bracket falls over the tab of the adapter, then close the trade again. Now the Connect Offroad is connected. Put the wheelchair off the brake and you can start driving. If there are any ambiguities for you, we refer you to the videos on our website: www.:Roam.nl

AFKOPPELEN

Disconnecting the Connect Offroad is also very easy. First put the wheelchair on the brake, then you have to stop the Connect Offroad, preferably grip the handlebars, then open the connector with your other hand and hold up the bracket of the clamp. Now you can close the lever of the connector again so that the bracket is on the tab. Then slowly slide the Connect Offroad out of the adapter and put it away. If there are any ambiguities for you, we refer you to the videos on our website: www.:Roam.nl

WARRANTY PERIOD:

You have 1 year warranty on the parts and 5 years on the frame.



Maintenance and your professional

Vital parts such as cables, spokes and brakes get a chance to "set" themselves in the start-up period. However, in order to be able to continue driving pleasantly and safely, a major annual turn is necessary. Your professional not only has the tools and experience to set and lubricate hidden rotating parts such as headset and hubs, but his hands, eyes and ears are also able to detect small imperfections in time. Before winter sets in or before a holiday trek, we advise you to have the bike checked. Proper maintenance is also extremely important with regard to the warranty. Make an appointment with your dealer before the service will be executed.

Maintenance Connect Offroad

1. Clean the Connect Offroad with a dry cloth.
2. In case of serious dirt, clean the Connect Offroad with a soft damp sponge and dry it.
3. Do not spray the Connect Offroad with a high pressure washer.
4. Do not apply chemical cleaning agents (chlorine/bleach/soda) when cleaning the Connect Offroad
5. Lubricate aluminium and chrome parts with an acid-free petroleum jelly or special grease (ask your dealer).
6. Store the Connect Offroad dry.

Tools intended for dealer/Roam Specialist

For the modest work, a simple set of tools is sufficient. You are already well on your way with a set of wrenches or socket wrenches (8 to 15), Allen wrenches, screwdriver, combination pliers, a can of bicycle oil and of course a tire repair kit. When replacing, make sure that original or equivalent parts are assembled for better operation and because in some cases the warranty will otherwise expire. Ask your Roam dealer for advice!



Check regularly:

1. Are all bolts and nuts secure?
2. Is the tread of the tires not worn out and do they not have weak spots or bumps?
3. Are the tires on pressure (tire pressure is on the side of the tire)?
4. Is the side reflection of the tires clean and therefore clearly visible in the dark?
5. Is the quick release working properly? (there must be no play on it, nor must the quick release be loose).
6. If any doubt, contact your dealer.

Tightening moments intended for dealer/Roam Specialist

Below you will find an overview of recommended tightening moments. We recommend that you tighten the listed parts with a torque wrench based on the values mentioned. Tightening the parts harder can lead to damage to your bike. It is best to ask your Roam dealer for help and advice for this action.

Steering plug bolt 12-15 Nm

Expander bolt with angled bobbin 20-23 Nm

Hub axle nuts front wheels 20-27 Nm



Guarantee

Roam gives a 5-year warranty against defects in material and finish on the frame. All parts and components manufactured and marketed by Roam have a 1-year warranty. This warranty does not include: tire and parts wear. Roam will repair or replace parts where necessary, provided that an official Roam dealer has agreed to return within the established warranty period. Roam does not guarantee parts that are damaged as a result of incorrect assembly and / or repair by the relevant owner, neglect, accident or incorrect use.

Contact

ROAM Special Cycles B.V.
Buitenkamp 20
5321KB Hedel
Tel.nr. 073-5997042
E-mail: info@roam.nl
Internet: www.roam.nl



EU-CONFORMITEITSVERKLARING (EU CONFORMITY DECLARATION)

Roam Special Cycles
Buitenkamp 20
5321KB Hedel
E-mail: info@**roam**.nl.

SRN (Single Registration Number): APP000006468

Verklaart dat ondervermeld Roam Product:
Declares that the below stated Roam Product:

Product: (Mention the product)
Connect Offroad

CE Markering: (CE Marking)

CE

Omschrijving Product:
(Describe the product)

- 1. een actief hulpmiddel dat verlichting, verbetering of ondersteuning geeft van lichamelijke immobiliteit handicaps of beperkingen.**
 - 2. het verruimen/verbeteren van het zoveel mogelijk zelfstandig functioneren.**
-
- 1. an active device that provides relief, improvement or support for physical immobility, disabilities or limitations.**
 - 2. expanding/improving independent functioning as much as possible.**

Udi-nummer (Unique identification number): **870261760Coo97899975321YL**

Risicoclassificatie **1** (conform de classificatie MDR 2017/745 EU).

Risk classification **1**: (according classification MDR 2017745 EU).

Voldoet aan de volgende (wettelijke)Regelgeving:

Comply with the below stated European regulation.

- **MDR (Medical Device Regulation)(2017/745/ EU)**
- **Machinerichtlijn(2006/42/EG) - Machinery Directive (X)**
- **EMC richtlijn(2014/30/EU) - Electric Magnetic Compatability Directive(X)**
- **Laagspanningsrichtlijn(2014/35/EU)- Low voltage Directive (X)**

Datum (Date): **1 January 2023**

Handtekening (Signature):

Roam Special Cycles
Johan de Rouw, Managing Director/JDR

Note: (X) niet toepasbaar/not applicable